



HEXIWASH



HYGIENIC HAND WASH

ACTIVE INGREDIENTS

2% Chlorhexidine gluconate;
0.5% Emollients

PRODUCT CHARACTERISTICS

Ready-to-use; Fragranced; pH balanced;
Non-toxic; Non-staining
Non-caustic; Biodegradable

RECOMMENDED USES

Hand and skin cleaning and disinfection.
As recommended by the CDC & WHO.

PPE REQUIREMENTS

None strictly necessary, however OSH requires the use of eye protection & good ventilation when mixing, handling or discarding chemical disinfectants.



For additional information about this product, or to give product feedback to the manufacturer, please scan this QR code.

PRODUCT DESCRIPTION

INSTRUCTIONS FOR USE

Thoroughly wet hands first, in order to achieve maximum foaming capacity. Apply one pump (approximately 3mL) of undiluted **PrimeSafe HexiWash** onto wet hands. Work into a lather & spread over hands and wrists, ensuring a contact time as per your required efficacy, see efficacy table below right. Ensure that the appropriate hand washing technique is used for your environment. Rinse well under running water. Dry hands thoroughly on a clean, disposable towel. Repeat between every patient, after eating and using the bathroom.

Available sizes:
25L, 5L, 800mL sachet; 500mL Lotion Pump.

Dilution Factor:

Ready to use

Physical Form:

Pink liquid

Odour:

Indistinct chemical smell

pH:

5.0-6.0

Risk Classification: B / IIa

GMDN Code: 41551

GMDN Descriptor: Liquid Skin Soap

Incompatibility:

Do not mix with other chemicals



EFFICACY TESTING:



	Tested To:	Time: (mins)	Dilution:	Conditions:
Bactericidal Efficacy	SANS 51276 (EN 1276): Enterococcus hirae, Escherichia coli, Pseudomonas aeruginosa & Staphylococcus aureus	1	NEAT	Dirty
	EN 1499 (Hygienic Hand wash)	3		
Fungicidal & Yeastical Efficacy	SANS 51650 (EN 1650): Aspergillus brasiliensis & Candida albicans	3	NEAT	Dirty
Sporocidal Efficacy	SANS 53704 (EN 13704): Bacillus subtilis	3	NEAT	Dirty
Dermal Irritation & Corrosivity	OECD 439 EN 1499 (Hygienic Hand wash)	3	NEAT	Dirty