



VACUFLUSH



For additional information about this product, or to give product feedback to the manufacturer, please scan this QR code.

**IN-SITU VACUUM SYSTEM LINE
FLUSH, DESCALER & DISINFECTANT**

ACTIVE INGREDIENTS

15% Citric Acid; 5% Solvent;
5% Surfactant

PRODUCT CHARACTERISTICS

Concentrated; Unfragranced;
Acidic; Non-staining
Non-caustic; Biodegradable

RECOMMENDED USES

In-situ terminal cleaning, disinfection, descaling & odour control of healthcare suction systems.

Recommended for weekly maintenance of suction systems where no other product is used.

Recommended for monthly descaling where a daily maintenance product is used.

PRODUCT DESCRIPTION

INSTRUCTIONS FOR USE

Dilute with hot water (>50°C) at 12.5% [125mL, or 5 pumps with a dosing pump, of PrimeSafe VacuFlush with 1L of water], immediately prior to use. Aspirate the solution into the suction lines, and circulate for 15 minutes. Flush the system with water thereafter. Repeat at a minimum of once per week, and especially when the system undergoes a deep clean. Dilute only enough for immediate single use – if unused, discard immediately.

PPE REQUIREMENTS

Latex or nitrile gloves and chemical splash goggles, however OSH also requires the use of good ventilation when mixing, handling or discarding chemicals.

Available sizes:

5L Dosing Pump, 1L Dosing Bottle

Dilution Factor:

12.5% (125mL per L)

Physical Form:

Translucent clear liquid

Odour:

Mild chemical odour

pH:

2-3

Risk Classification: B / IIa

GMDN Code: 47631

GMDN Descriptor:

Medical device disinfection agent.

NRCS Registration:

NRCS/8054/242613/286

SAHPRA Medical Device Registration:

Awaiting Class Registration Call-up

Incompatibility:

Not recommended for use directly on or with plated or non-stainless steel handpieces



EFFICACY TESTING:



	Tested To:	Time: (mins)	Dilution:	Conditions:
Bactericidal Efficacy	SANS 53727 (EN 13727): Enterococcus hirae, Pseudomonas aeruginosa & Staphylococcus aureus	15	12.5%	Clean
Corrosivity	SANS 636 (In-house): Non-corrosive.			