



# WOUND PROTECTION GEL

*Hydrogel-based liquid plaster*

## ACTIVE INGREDIENTS

**Vetguard Wound Gel** contains 2% Gelling agents, 0.5% emollients, 1.25% SteriTech Concentrate (a QAC/TBTO blend); 95.75% Inert Ingredients



## QUICK

Dries to form a 'second skin' in just 5-10 minutes.

## EASE OF USE

Application and removal does no damage to wound bed.

## ECONOMICAL

Use of product negates the need for bandages, plasters and dressings.

## SAFE

Non-toxic, non-staining, non-caustic and biodegradable.

## BITTER TASTE

Discourages licking of the area by the animal

## ACCELERATED HEALING

Maintains moisture level of wound bed, resulting in ideal conditions for healing and decreased scar tissue formation.

## PPE REQUIREMENTS

None necessary, however caution should be taken to avoid direct, extended eye contact with the product.

## INSTRUCTIONS FOR USE

Apply directly from the container to a clean, non-necrotic wound site. Spread into as thin a layer as possible using a gloved finger, gauze or spatula. Allow 5-10 minutes for the product to dry into a skin-like layer. Do not cover the treated area with a bandage or plaster. To remove, wet the area & wipe lightly with a clean cloth, cotton wool or gauze. Reapply daily or after washing or bathing. For external use only.

## RECOMMENDED USES

Used for dressing wounds in hard-to-reach areas. Recommended for Hot spots, bites, burns, debrided wounds, fragile skin, roasties and wounds with hard to remove fur.

## FIRST AID



**In case of eye contact:** Flush eyes thoroughly with water. Redness may occur, but should dissipate within 30 minutes. Should it persist, consult your doctor.

**In case of ingestion:** Do not induce vomiting, rinse out mouth then drink a large glass of water, and consult your doctor.

**In case of skin contact:** Rinse affected area with water.

**In case of inhalation:** Remove patient to fresh air for 15 minutes.



**SAMED** member company  
south african medical device industry association

## PRODUCT DESCRIPTION

### Available sizes:

250mL Push Disc, 5mL sachet

### Dilution Factor:

Ready to Use

### Physical Form:

Translucent blue paste

### Odour:

Mild chemical smell

### pH:

5-6.5

### Risk Classification:

B / II

### GMDN Code:

47763

### GMDN Descriptor:

Wound Hydrogel Dressing, Non-sterile

### Act 36 of 1947 Reg. no.:

G3766

### Incompatibility:

Anionic detergents, Natural Soaps, Strong oxidisers

## PRODUCT TESTING

|                       | Tested To:  |
|-----------------------|---|
| Bactericidal Efficacy | SANS5261  |
| Oral Toxicity         | Northview Pacific Lab, Inc. - 40 CFR part 796, 797, 798 |
| Inhalation Toxicity   | Leberco Testing, Inc. - 40 CFR part 163.81              |
| Dermal Toxicity       | Northview Pacific Lab, Inc. - 40 CFR part 796, 797, 798 |
| Eye Irritation        | Northview Pacific Lab, Inc. - 40 CFR part 796, 797, 798 |